

ifill



Fill up... and go!

 **DeVilbiss**
HEALTHCARE



The DeVilbiss iFill® Personal Oxygen Station represents the latest in portable oxygen convenience, providing unmatched flexibility and ease of use. The iFill unit extracts purified oxygen from room air using oxygen concentrator technology and pressurises it to fill oxygen cylinders.

Cylinders are filled safely, quickly and conveniently in your home. Filling the cylinder is incredibly easy. A custom designed push-click connector allows for a simple, safe and secure connection between the iFill and cylinder. Once connected correctly simply press the "start/stop" button and the control panel lights indicate when the cylinder is filling and when the cylinder is full.

An ML6 (1.2 l) iFill cylinder can be filled in 90 mins. If the cylinder is fitted with a DeVilbiss Pulse Dose® Oxygen conserver this cylinder will last for approximately 4 hours before needing to be refilled. The iFill is compatible only with the DeVilbiss iFill Cylinders. A wide range of cylinder sizes are available to suit the needs of the ambulatory user. If you already have a Oxygen Concentrator, you don't have to change it, the DeVilbiss iFill Personal Oxygen Station sits along side it. The iFill can be placed and used virtually anywhere in the home, minimising the noise and disruption to daily life.

Features and benefits

- Cylinders can be refilled as often as you need
- No more waiting for your cylinders to be delivered
- iFill has the ability to top up half full cylinders
- Wide range of cylinder sizes available
- Cylinders available with Pulse Dose conserver
- Safe and easy to use



iFill with connected Cylinder



Adapter for different cylinder sizes



Easy to operate click-on/off system



Simple push-button set up

SPECIFICATIONS

Dimensions (H x W x D)	72.5 x 57.0 x 31.0 cm	Type	content litre	weight kg	filling time* minutes	usage time* PD (h) CF
Weight	30.0 kg	ML6	1.25	2.18	90	4.3 1.4
Electrical Requirements	230 V, 50 Hz, 1.7 Amp	C	1.80	2.54	130	6.1 2.0
O ₂ -Concentration	93% ± 3%	D	2.90	3.27	215	10.5 3.5
Average Filling Pressure	2,000 ± 100 psig	E	4.70	4.45	350	17.2 5.7

* Note: All filling times are approximate and may vary depending on environmental conditions, such as altitude.

Distributor:



CE 0044

Sunrise Medical Limited
High Street Wollaston/Stourbridge
West Midlands DY8 4PS
England

Phone + 44 (0) 13 84 44 68 58
Fax + 44 (0) 13 84 44 66 28
www.devilbisshealthcare.com
enquiries@devilbisshc.com